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Rollator Aluminium

Art.-Nr. 11447 000

PZN 04608572

HMV-Nr. 10.50.04.1201

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Rollator aluminium

Item No. 11447 000

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

Indication/Contraindication

The rollator is a mobility aid for people with mobility restrictions, e. g. people with a disability that makes walking difficult or who cannot walk for long, providing them with the opportunity to go further afield. Staying on the move with a rollator not only serves to improve circulation, coordination and balance, but it also enhances the user's well-being.

This rollator is designed both for use in enclosed areas and in the open. It is equipped with a removable basket and a tray. The rollator provides a seat and back rest for short rests. It can be folded easily for transportation, thus saving space. The rollator should not be used by persons with perception impairments, serious problems with their balance or an inability to sit. The user must have adequate perception skills and adequate physical ability to be able to steer a rollator.

Safety Indications

- Talk to your doctor or specialist retailer to see whether this Rollator is optimally suitable for you and your needs.
- Check that the Rollator is in perfect condition every time before you use it. Check the following components and make sure:
 - all screws, nuts and other parts are firmly tightened
 - all wheel surfaces are intact
 - the height adjustment screws are firmly tightened
 - the brakes function both as parking brakes and hand brakes
 - there are no visible signs of wear on the frames, welded joints and plastic elements
 - wheels and wheel forks should be free of dirt and turn smoothly.
 Do not use a damaged or defective Rollator!
- Ensure that the Rollator is correctly engaged before every use.
- Always activate the parking brake when you stop or park the Rollator. Always use the hand brakes slowly, evenly and both together.
- The braking effect can be reduced if the tyre surface is worn. Inspect the braking action of the brakes regularly.
- The Rollator should only be used on firm and even surfaces. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also limited. Please adapt the way you move and walk to these dangers.
- On smooth surfaces such as tiles, stone flooring, PVC etc. the wheels can block when braking under certain circumstances. Therefore please check the braking performance of your Rollator carefully on these floors and adjust the way you move and walk accordingly. If you wish to sit on the Rollator you should also make sure that the wheels have a sufficient hold on the surface when the brakes are applied and that the Rollator cannot slide away.
- The Rollator should not be used as a support when getting up from a crouching or sitting position!
- The maximum permissible load of the rollator is 120 kg. This includes the max. luggage load of 15 kg in the basket.
- The basket included in the scope of delivery is designed for a max. luggage load of 15 kg. The basket must not be overloaded. Only use the basket for transporting objects.
- The rollator must not be used outside its intended use or for transporting objects or persons. A maximum of 10 kg may be transported in the basket provided for this purpose.
- Ensure that the hand grips are set to the correct height for the user in order to avoid incorrect loads or falls. Always hold both handles, otherwise you could lose your balance. Walk between the hand grips. When walking, be careful not to bump your feet against the wheels or step assistance.
- Make sure that your hands or clothes do not come into contact with the moving parts of the Rollator! Be particularly careful when folding and unfolding the Rollator. This applies in particular to hinges under the seat or in the middle of the cross brace.
- The rollator should be used on the pavement. The rollator must not be used in moving traffic. For your personal safety in road traffic, it is recommended to wear additional reflectors on your clothing in addition to the reflectors already attached to all four sides of the rollator.
- Do not sit on the rollator when travelling on public transport. Park the rollator with the brakes on so that it cannot fall over or roll away. Look for a seat close by.
- If you have to cross (tram) tracks, drive over them at a right angle and as quickly as possible. at a right angle and cross them quickly. Make sure that the wheels do not get caught in the rails. The wheels can get wedged in grids, cracks and uneven surfaces. Avoid all obstacles wherever possible.
- Be careful near children. Children should not play with the Rollator!
- Do not hang any handbags or carrier bags on the handles, the balance of the Rollator can be impaired if you do.
- Do not carry out any adjustments or modifications, which impair the safety of the design. Parts such as wheels, handles or brake cables may only be replaced by a specialist retailer.
- Only use accessories approved by the manufacturer. Please also read the instructions for use for each accessory.
- Do not carry or pull the Rollator by the brake cables.
- In strong sunlight, parts of the rollator, e.g. handles or seat, can heat up. In these cases, skin contact should be avoided to prevent burns. Make sure to cover your rollator or park it in a shady place.
- Please prevent skin contact with the Rollator if the Rollator has been in cold temperature for a long period of time in order to prevent freezing.
- Keep the rollator away from naked flames.
- Please prevent running into door frames, furniture, etc. with the Rollator and check the secure position of the Rollator when parked. The frame, connection elements and welding seams may be damaged if the Rollator tips over.
- Please also prevent placing stress on the Rollator when folded together! This could damage the frame, connection elements and welding seams.
- Do not use the Rollator in stairs or escalators.
- Do not stand on the rollator.
- Do not lean on the brake lever.
- If you have any queries, please refer to your dealer.

Remove all packaging material. Check the product has not been damaged in transit before using it for the first time. Pay particular attention to ensuring that the wheels and wheel suspension, screw and plastic connections and brake cables are intact. The rollator is delivered pre-assembled. Before using for the first time, check that the brakes work properly; they may need to be readjusted. The hand grips must be adjusted to the appropriate height for the user. Please dispose of the packaging material correctly.

Included

Rollator, backrest, tray, basket, cane holder


Parts:

- | | | |
|-------------------|-----------------------------------|---------------------------|
| 1. brake lever | 6. back wheel with braking device | 10. seat |
| 2. Security screw | 7. handlebars | 11. Reflectors |
| 3. Tray | 8. backrest | 12. basket |
| 4. cane holder | 9. clips | 13. front wheel steerable |
| 5. brake pad | | |

Assembly

- Position the rollator in front of you and press the seat downwards until the folding catch engages (pic. 1). If the brake levers are locked into position release these first. Guide the handlebars with the handles into the frame tube (pic. 2). At the same time make sure that the brake cables are not twisted.
- Adjust the handles to the right height for the user. They can be adjusted by sliding the handlebars in and out. When the user is standing upright the handles should be adjusted to the height of their wrists.
- In order to fix the handles in place insert the locking screw through the frame tube from the inside outwards. The screw head fits into the opening on the frame. Place the washer onto it and tighten the star screw (pic. 3+4).
- You can insert the back rest into the sockets in the tube on the front of the rollator. You can fix it with the enclosed clips (pic. 5-8).

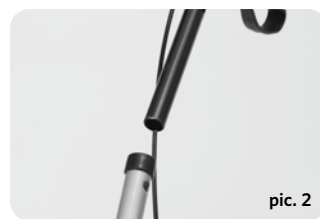5. You can hang the basket with the hooks on the crossbar underneath the seat. The underneath of it rests on the front crossbar (pic. 9).
6. Insert the tray with the little »feet« underside in the holes of the seat (pic. 9).

Use

Unfold/fold



- When opening out and when folding the rollator, make sure that no clothing or parts of the body become trapped. To unfold, press the seat downwards until the folding catch engages (pic. 10).
- To fold up, first secure the brakes as described under »Braking function«, so that the rollator cannot roll away. Remove the basket and tray. Release the folding catch under the seat by holding it through the grip opening in the seat. At the same time pull the red lever underneath the seat and pull both upwards. The rollator then folds up. The rollator is folded up completely when the front and back wheels are almost parallel to each other (pic. 11).

Brake

The built-in brake functions as an operating brake for slowing down when moving. The parking brake can be used when the rollator is stopped.


1. Speed is reduced by pulling the brake lever upwards (operating brake) (pic. 13).
2. The parking brake is activated by pressing the brake lever down until it engages (pic. 12).
3. The parking brake is released from its locked position by pulling the brake lever upwards.
4. The rollator should not be pushed while the operating or parking brakes have been applied. If the rollator is pushed while the operating or parking brakes have been applied, the tyres and brakes may be subject to excessive wear and deteriorate.
5. Regularly check that the brakes work satisfactorily. We recommend checking before every use. The rollator must not be used with damaged brakes or damaged wheels.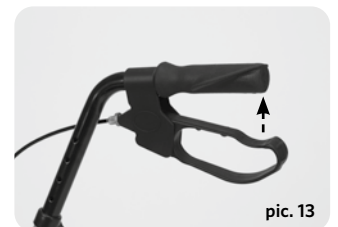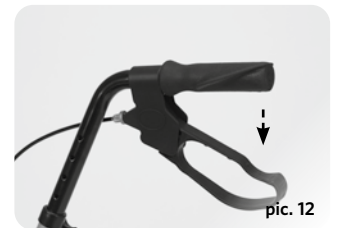

Brake adjustment (pic. 14)

If the braking force is decreasing due to wear on the wheels, the brake shoe can be adjusted. The brakes must be adjusted so that both wheels brake evenly. The distance between the brake pad and the wheel surface should be about 2-3 mm. After replacing wheels, the brake setting must be checked again and if necessary, readjusted.

1. Loosen the locking nut.
2. Adjust the braking force by turning the brake cable spanner inwards or outwards.
3. Check the braking function by turning the wheels and operating the brake levers. The wheels should turn freely but stop immediately when the brakes are applied. The wheels must not be able to move when the brakes are applied.
4. Tighten the locking nut firmly again.
5. If you are unsure about the correct adjustment of the brakes, please contact your specialist retailer.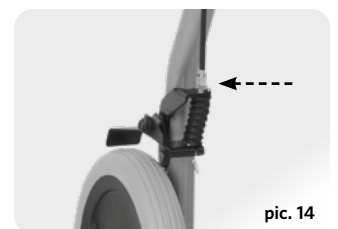

Basket

The basket allows the user to transport his shopping conveniently. You can hang it up using the hook on the crossbar underneath the seat. The underneath of it rests on the front crossbar (pic. 9).

Warning: Only use the basket to transport objects and do not overload it (max. 15 kg)!

Tray

The tray is designed only for small objects. The little »feet« under the tray enable it to be inserted securely in the holes of the seat. Do not place hot, heavy or sharp objects on the tray!

Warning: Do not use the tray as a seat!

Back rest

You can insert the back rest into the sockets in the tube on the front of the rollator. You can fix it with the enclosed clips (pic. 5-8).


The back rest is for support when sitting and facilitates comfortable sitting posture during a rest. Always lean slowly and evenly against it. Do not hang any coats or bags over it.

General care and cleaning

Rollator

The rollator can be cleaned with lukewarm water, neutral commercial and household cleaning agents and a soft cloth to remove dirt, then rinsed and dried. No water must get into the pipes. After cleaning or after rain, dry the walker thoroughly with a dry, clean cloth. Also clean the brake pads and springs regularly. Do not use abrasive, corrosive or bleaching cleaning agents. Do not use metal brushes or other sharp-edged cleaning aids for cleaning. Further cleaning instructions for reuse and before changing patients can be found under the Hygiene Plan.

Do not use the rollator in temperatures below -35°C or above $+60^{\circ}\text{C}$.

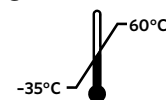Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines. Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary. If you find the brake action to be faulty in any way, please contact your dealer immediately.

Please find below a maintenance table and hygiene plan for the regular care and maintenance of this device:

Maintenance Table	
When	
Before every re-use	
What	Caution
Check general condition Visually inspect all frame parts including welded joints for deformation, damage, wear or corrosion	If you find any defects, replace the affected parts of the device.
Check hand grips visually and by attempting to turn them	If you find that the hand grips are damaged or no longer firmly connected to the frame, they must be replaced.
Check the tyre tread and ball bearings	If you find that the wheels are worn or cracked, they must be replaced.
Check the folding lock	If the folding mechanism no longer works properly, it must be repaired or replaced.
Check the connecting elements for tightness	If you find that screws, nuts, rivets, etc. are loose or missing they must be tightened, fitted or replaced.
Check the braking system is working satisfactorily Apply the operating brake until it reaches the stopping point. The wheels must not be able to turn now when pushed hard. Apply the parking brake by pressing the brake levers down until it engages. The wheels must not be able to turn now when pushed hard.	If the brakes are defective, the brake slack, brake cables, brake shoes and wheels must be checked and the defective part of the device must be replaced if necessary.

Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary. If you find the brake action to be faulty in any way, please contact your dealer immediately.

Hygiene Plan

The rollator must be cleaned and disinfected before each re-use or before each change of patient. The disinfection of the rollator must only be carried out by a specialist.

Cleaning/disinfection

1. Put on protective gloves and, if necessary, protective clothing (disposable apron, mouth and nose protector, etc.) before preparing the rollator.
2. Always clean first, then disinfect.
3. To remove any dirt, the rollator can be cleaned with lukewarm water, neutral, commercial and household cleaning agents and a soft cloth, then rinsed and dried. No water must get into the pipes. Also clean the brake pads and the springs. Do not use abrasive, corrosive or bleaching cleaning agents. Do not use metal brushes or other sharp-edged cleaning aids for cleaning. After using cleaning agents, rinse the rollator thoroughly with clean water. Before disinfecting, there must be no residues of cleaning agents on the surface. After cleaning, dry the rollator with a dry, clean cloth.
4. Wipe or spray the rollator with a disinfectant listed by DGHM/VAH (follow the manufacturer's instructions). Disinfectants containing chlorine or phenol as well as vinegar-based cleaning agents must not be used. Wipe disinfection is always preferable to spray disinfection. After the disinfectant has taken effect, dry the rollator with a dry, clean cloth. After cleaning, dry the rollator off with a clean, dry cloth.

Storage

Store the Rollator light and dust-protected in a dry place. Do not store near a heat source. Protect the rollator from dirt and contamination after treatment or if it will not be used for a long period of time.  

Disposal

Do not dispose of the rollator in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

Technical data

Material:

- Frame: aluminium (powder coated)
- Wheels: Thermoplastic rubber (TPR)
- Dimensions: 63 x 67 cm/63 cm (with basket/with backrest) x 82,5-93,5 cm (W x L x H)
- Folding measure: Width: ca. 63 cm
Length: approx. 50 cm (with backrest)/approx. 28 cm (without backrest)
Height: approx. 95-108 cm
- Wheels: 19,3 x 3,1 cm
- Seat level: 55 cm
- Distance between handles: 43,5 cm
- Weight: 6,5 kg (without accessories)
Basket: 0,6 kg; Tablett: 0,3 kg; Backrest: 0,3 kg
- Turning cycle: 84 cm
- Max. loading capacity: 120 kg

Basket

- Material: coated wire
- Dimensions: 40,5 x 26,5 x 17 cm (W x L x H)
- Max. loading capacity: 15 kg

 This device complies with Regulation (EU) 2017/745  on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the national competent authority in your country.

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Fabrikant: Ludwig Bertram GmbH
Albo-Healthcare GmbH